




## Instructions for Use

### EcoVue® & EcoVue® HV Sterile Ultrasound Gel

REF	Product Description
280	EcoVue® Ultrasound Gel 20g SafeWrap™ Packet (sterile)
380	EcoVue® HV Ultrasound Gel 20g SafeWrap™ Packet (sterile)
280NW	EcoVue® Ultrasound Gel 20g Packet (sterile)
380NW	EcoVue® HV Ultrasound Gel 20g Packet (sterile)
282	EcoVue® Ultrasound Gel 32g SafeWrap™ Packet (sterile)
282NW	EcoVue® Ultrasound Gel 32g Packet (sterile)

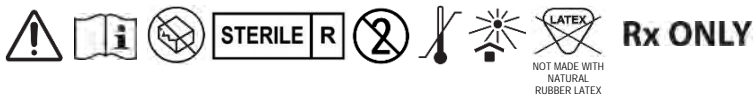

**HR PHARMACEUTICALS INC.**  
 2600 Eastern Boulevard, Suite 201  
 York, PA 17402 USA  
 (877) 302-1110



**EC REP** **EMERGO EUROPE**  
 Prinsessegracht 20  
 2514 AP The Hague  
 The Netherlands

For more information on symbols, visit [www.ecovue.com](http://www.ecovue.com).



### INDICATIONS FOR USE

EcoVue® and EcoVue® HV Sterile Ultrasound Gel may be used during ultrasound procedures on all patient populations where sterility is indicated or otherwise desired, or during procedures where cross contamination is a concern.

### DESCRIPTION

EcoVue® and EcoVue® HV Sterile Ultrasound Gel is a sterile, non-irritating, non-sensitizing, non-staining and water-soluble scanning gel intended for use during sterile ultrasound procedures. The gel is an accessory used to couple sound waves between a patient and medical imaging equipment during ultrasound procedures in a healthcare setting. EcoVue® and EcoVue® HV Sterile Ultrasound Gel may be used with all commercially available ultrasound devices.

### CONTRAINDICATIONS

None.

### WARNINGS

- If irritation occurs, discontinue use and consult a physician. **NOTICE:** Any serious incident that has occurred in relation to this device should be reported to HR Pharmaceuticals.
- EcoVue® and EcoVue® HV Sterile Ultrasound Gel is intended for single-use only. Re-use could result in cross-contamination, risk of infection, or adverse reaction for the patient.

### PRECAUTIONS

Discard and do not use EcoVue® or EcoVue® HV Ultrasound Gel if package is damaged or opened.

### PRE-USE, HANDLING & INSPECTION

- EcoVue® and EcoVue® HV Sterile Ultrasound Gel should be thoroughly inspected prior to use to ensure packaging is not damaged or defective.
- To reduce the risk of cross-contamination, standard practices for sterile procedures should be used.

### INSTRUCTIONS FOR USE

- Remove SafeWrap™ outer pouch (if applicable).
- Manually open the packet by tearing along the perforations where indicated.
- Apply the desired amount of EcoVue® or EcoVue® HV directly to the patient or probe as needed.
- Discard packet after use.

### RECOMMENDED STORAGE CONDITIONS

Store in a clean, dry place at room temperature (15° - 30°C; 59° - 86°F) and away from direct sunlight.



## Instructions for Use

### EcoVue® & EcoVue® HV Non-Sterile Ultrasound Gel Single-Use Packets

REF	Product Description
281	EcoVue® Ultrasound Gel 20g Packet (non-sterile)
381	EcoVue® HV Ultrasound Gel 20g Packet (non-sterile)
283	EcoVue® Ultrasound Gel 32g Packet (non-sterile)
383	EcoVue® HV Ultrasound Gel 32g Packet (non-sterile)



**HR PHARMACEUTICALS INC.**  
 2600 Eastern Boulevard, Suite 201  
 York, PA 17402 USA  
 (877) 302-1110  
 www.ecovue.com



**EC REP** **EMERGO EUROPE**  
 Prinsessegracht 20  
 2514 AP The Hague  
 The Netherlands

For more information on symbols, visit [www.ecovue.com](http://www.ecovue.com).



#### INDICATIONS FOR USE

EcoVue® and EcoVue® HV Non-Sterile Ultrasound Gel Single-Use Packets are intended for use in facilities where cross-contamination is of concern on all patient populations during non-invasive ultrasound procedures on intact, unbroken skin.

#### DESCRIPTION

EcoVue® and EcoVue® HV Non-Sterile Ultrasound Gel Single-Use Packets are non-sterile, non-irritating, non-sensitizing, non-staining and water-soluble gels intended for use as a scanning gel in diagnostic and therapeutic ultrasound procedures. The gel is an accessory used to couple sound waves between the patient and medical imaging equipment during ultrasound procedures in a healthcare setting. EcoVue® and EcoVue® HV Non-Sterile Ultrasound Gel Single-Use Packets may be used with all commercially available ultrasound devices.

#### CONTRAINDICATIONS

None.

#### WARNINGS

- If irritation occurs, discontinue use and consult a physician. **NOTICE:** Any serious incident that has occurred in relation to this device should be reported to HR Pharmaceuticals.
- EcoVue® and EcoVue® HV Non-Sterile Ultrasound Gel Single-Use Packets are intended for single-use only. Re-use could result in cross-contamination, risk of infection, or adverse reaction for the patient.

#### PRECAUTIONS

Discard and do not use EcoVue® and EcoVue® HV Ultrasound Gel Single-Use Packets if package is damaged or opened.

#### PRE-USE, HANDLING & INSPECTION

EcoVue® and EcoVue® HV Non-Sterile Ultrasound Gel Single-Use Packets should be thoroughly inspected prior to use to ensure packaging is not damaged or defective.

#### INSTRUCTIONS FOR USE

- Manually open the EcoVue® and EcoVue® HV Non-Sterile Ultrasound Gel Single-Use Packet by tearing along the perforations where indicated.
- Apply the desired amount of EcoVue® directly to the patient or probe as needed.
- Discard packet after use.

#### RECOMMENDED STORAGE CONDITIONS

Store in a clean, dry place at room temperature (15° - 30°C; 59° - 86°F) and away from direct sunlight.



## Instructions for Use

### EcoVue® & EcoVue® HV Non-Sterile Ultrasound Gel Multi-Use Products

REF	Product Description
284	EcoVue® Ultrasound Gel 60g Tube (non-sterile)
285	EcoVue® Ultrasound Gel 250g FlexPac® (non-sterile)
286	EcoVue® Ultrasound Gel 250mL Bottle (non-sterile)
386	EcoVue® HV Ultrasound Gel 250mL Bottle (non-sterile)



**HR PHARMACEUTICALS INC.**  
 2600 Eastern Boulevard, Suite 201  
 York, PA 17402 USA  
 (877) 302-1110



**EC REP EMERGO EUROPE**  
 Prinsessegracht 20  
 2514 AP The Hague  
 The Netherlands

For more information on symbols, visit [www.ecovue.com](http://www.ecovue.com).



#### INDICATIONS FOR USE

EcoVue® and EcoVue® HV Non-Sterile Ultrasound Gel is intended for use in facilities where cross-contamination is of minimal concern on all patient populations during non-invasive ultrasound procedures on intact, unbroken skin.

#### DESCRIPTION

EcoVue® and EcoVue® HV Non-Sterile Ultrasound Gel is a non-sterile, non-irritating, non-sensitizing, non-staining and water-soluble gel intended for use as a scanning gel in diagnostic and therapeutic ultrasound procedures. The gel is an accessory used to couple sound waves between the patient and medical imaging equipment during ultrasound procedures in a healthcare setting. EcoVue® and EcoVue® HV Non-Sterile Ultrasound Gel may be used with all commercially available ultrasound devices.

#### CONTRAINDICATIONS

None.

#### WARNINGS

If irritation occurs, discontinue use and consult a physician. **NOTICE:** Any serious incident that has occurred in relation to this device should be reported to HR Pharmaceuticals.

#### PRECAUTIONS

- Discard and do not use EcoVue® and EcoVue® HV Ultrasound Gel if package is damaged or opened, or if tamper-evident seal is damaged or missing.
- Do not refill EcoVue® and EcoVue® HV Ultrasound Gel multi-use products.

#### PRE-USE, HANDLING & INSPECTION

- EcoVue® and EcoVue® HV Non-Sterile Ultrasound Gel should be thoroughly inspected prior to use to ensure packaging is not damaged or defective.
- For EcoVue® Non-Sterile Ultrasound Gel in the 250g FlexPac® (REF 285), visually inspect sides, bottom, and nozzle to ensure there is no gel leakage.

#### INSTRUCTIONS FOR USE

- Remove tamper-evident seal and replace cap (if applicable).
- Apply the desired amount of EcoVue® or EcoVue® HV directly to the patient or probe as needed. To avoid contamination, tips of containers should not come in contact with patient, staff or instruments.
- Close cap and store in a suitable place. Discard container when empty (or sooner, if required by facility internal procedures).

#### RECOMMENDED STORAGE CONDITIONS

Store in a clean, dry place at room temperature (15° - 30°C; 59° - 86°F) and away from direct sunlight.