

Acclimate® Gel Warmer

Operating Instructions



Model No. 289F, 289B, and 289S

Specifically designed for EcoVue® Ultrasound Gel products

READ INSTRUCTIONS BEFORE USE


1: what's included




(1) Acclimate® Gel Warmer



(1) Operating Instructions

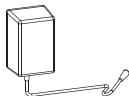

(1) Power Cord (12V)

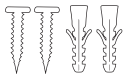

(2) Screws and (2) Anchors (for wall mounting)

2: PRECAUTIONS

1. Connect device to properly grounded outlets only.
2. Do not immerse in water.
3. Only use indoors.
4. Do not operate if the cord or plug is damaged in any manner.
5. Unplug device when not in use.
6. Allow unit to cool before cleaning.
7. Do not use secondary attachments on device. Use appropriate manufacturer parts only.
8. Only use device for its intended use.
9. Return damaged devices to the manufacturer.
10. If instructions are misplaced, visit <https://ecovue.com/>.

3: how to assemble

setup

1. Place Acclimate® Gel Warmer in the upright position on a smooth and stable surface away from any hazard that may inadvertently come in contact with the device. The device may be wall mounted.
2. Only operate device while it is in the upright position.
3. Plug female end of power cord into the back of the device.
4. Plug male end of power cord into an appropriate electrical outlet. Voltage information is listed on the back of the device.



wall mounting

Tools Required:



Phillips-Head Screwdriver



Level



Drill (drywall mounting)



Hole 1

Template for drywall mounting



Hole 2

1. The Acclimate® Gel Warmer may be wall mounted in a safe location to avoid damage to the device and/or wall.
2. Using a level and the above screw-hole template, place two marks on the wall.



CAUTION: Do not install screws where they may come in contact with wires or plumbing.

3. When mounting in wood, wall anchors are not required. Install screws at the two marks created in step two. Allow clearance between the head of the screw and the wall.
4. When mounting in drywall, pre-drill holes at the two marks created from step two. Insert wall anchors and install screws. Allow clearance between the head of the screw and the wall.
5. Position the device against the wall so the screws enter the large opening of the key holes on the back of the device.
6. Slide the device down to create a firm mount.

4: how to operate

1. When the Acclimate® Gel Warmer is plugged in, the temperature display segments and the red-green-blue LEDs will quickly cycle through during power up. The Tear Drop button will turn solid blue, indicating power and Stand-by Mode. When the warmer is in Stand-by Mode it does not heat.
2. To warm, press the Tear Drop button and release. The device will begin active warming mode, which can take approximately 1 hour. The Tear Drop button will turn green and flashes during the entire active warming cycle.
3. To change temperature, select the (+) or (-) arrows to toggle between 95°F and 105°F temperature range. Once unit reaches selected temperature, the Tear Drop button will turn from flashing green to solid green.

By default, the device is set to 100°F when turned on. If the (+) or (-) are not pressed to select new temperature between 95° - 105°F, then unit will heat up to 100°F.



NOTE: Acclimate® Gel Warmer may take 1.5-2 hours to fully heat EcoVue® Ultrasound Gel.



CAUTION: Sample a small amount of gel to test temperature prior to use.

4. To return the device to Stand-by Mode, press the button once. Always unplug device when not in use.

5: safeguards

The Acclimate® Gel Warmer is equipped with various feedback loops to prevent the device from overheating (109°F). If the device detects a failure in any of the loops, the Tear Drop button will turn red and remain in Stand-by Mode.



CAUTION: If the device ever overheats, do not continue use. See Warranty section.

6: technical specifications

Supply Voltage:	110V - 120V, S0/60Hz 220V- 230V, 50/60Hz (CE)
Power Rating:	1.0A from wall socket

7: how to service and clean

The Acclimate® Gel Warmer does not require routine maintenance. To clean, unplug the device and allow it to cool down. Use a standard disinfectant wipe to wipe the device down. Do not attempt to disassemble the device and clean internal electrical components.

8: warranty

Acclimate® Gel Warmers come with a two-year Limited Warranty from the date of purchase. Do not tamper with or try to service the device yourself for it will void the warranty.

All devices must be registered within 30 days of purchase. To register, complete the form <https://ecovue.com/acclimate-product-registration/>.

If you have any warranty or service questions, please contact HR Pharmaceuticals, Inc. at service@hrpharma.com.

9: additional information

For additional information on the Acclimate® Gel Warmer, please visit <https://ecovue.com/products/acclimate/>.

Declaration of Conformity

HR Pharmaceuticals, Inc. certifies this device was manufactured in accordance to the published specifications.

Manufacturer's Name: HR Pharmaceuticals, Inc.
Manufacturer's Address: 2600 Eastern Blvd, Suite 201
York, PA 17402
Device Name: Acclimate® Gel Warmer
Models: 289F, 289B, 289S

The product listed conforms to the following Directives and Standards:

Directive/Standard	Certified By
EMC Directive 2014/30/EU	METLabs, Inc.
EN 55014-1:2006/A1:2009/A2:2011	
EN 55014-2:1997/A1:2008	
EN 61000-3-2:2014	
EN 61000-3-3:2013	
US/FCC: Part 15, Subpart B	Underwriters Laboratory
ICES-003, Issue6, April 2017, Class B	
UI-499 Standard	