



## Non-Sterile Ultrasound Gel

### HV Non-Sterile Ultrasound Gel

## Instructions for Use

### REF

281	20 g Packet
283	32 g FlexPac® Pouch
284	60 g Flip-Top Tube
285	250 g FlexPac® Pouch
286	250 ml (8.5 oz.) Bottle

381	20 g Packet
383	32 g FlexPac® Pouch
386	250 ml (8.5 oz.) Bottle

### DESCRIPTION

EcoVue® Non-Sterile Ultrasound Gels are non-sterile, non-irritating, non-sensitizing, non-staining and water-soluble gels intended for use as a scanning gel in diagnostic and therapeutic ultrasound gel procedures. The gels are an accessory used to couple sound waves between the patient and medical imaging equipment during ultrasound procedures in a healthcare setting. EcoVue Non-Sterile Ultrasound Gels may be used with all commercially available ultrasound devices.

### INDICATIONS FOR USE

EcoVue Non-Sterile Ultrasound Gel Single-Use Packets (REF # 281, 283, 381, and 383) are intended for use in facilities where cross-contamination is of concern on all patient populations during non-invasive ultrasound procedures on intact, unbroken skin.

EcoVue Non-Sterile Ultrasound Gel Multi-Use Products (REF # 284, 285, 286, and 386) are intended for use in facilities where cross-contamination is of minimal concern on all patient populations during non-invasive ultrasound procedures on intact, unbroken skin.

### CONTRAINDICATIONS

None.

### WARNINGS

- If irritation occurs, discontinue use, and consult a physician. NOTICE: Any serious incident that has occurred in relation to this device should be reported to HR Pharmaceuticals and the competent authority in Member State where user is established.
- EcoVue Non-Sterile Ultrasound Gel Single-Use Packets are intended for single-use only. Re-use could result in cross-contamination, risk of infection, or adverse reaction for the patient.
- Caution: Federal law restricts this device to sale by or on the order of a physician or medical practitioner.

### PRECAUTIONS

- Discard and do not use EcoVue Non-Sterile Ultrasound Gel if package is damaged or opened.
- Discard and do not use EcoVue Non-Sterile Ultrasound Gel if tamper-evident seal is damaged or missing (REF # 284 and 285 only).
- Do not refill EcoVue Non-Sterile Ultrasound Gel Multi-Use Products (REF # 284, 285, 286, and 386).

### PRE-USE, HANDLING & INSPECTION

- EcoVue Non-Sterile Ultrasound Gel should be thoroughly inspected prior to use to ensure packaging is not damaged or defective.
- For EcoVue Non-Sterile Ultrasound Gel 250 g FlexPac Pouch (REF # 285), visually inspect sides, bottom, and nozzle to ensure there is no gel leakage.

### GEL WARMING

EcoVue Non-Sterile Ultrasound Gels, in any packaging format, may be used at room temperature or warmed in an ultrasound gel warmer, such as the Acclimate® Gel Warmer. Refer to the gel warmer manufacturer's operating manual/user manual for warming instructions.

### STEP-BY-STEP INSTRUCTIONS

#### Single-Use Packets (REF # 281, 283, 381, and 383)

1. Manually open the EcoVue Non-Sterile Ultrasound Gel Single-Use Packet by tearing along the perforations where indicated.
2. Apply the desired amount of EcoVue directly to the patient or probe as needed.
3. Discard packet after use.

#### Multi-Use Products (REF # 284, 285, 286, and 386)

1. Remove tamper-evident seal and replace cap (REF # 284 and 285 only).
2. Apply the desired amount of EcoVue directly to the patient or probe as needed. To avoid contamination, tips of containers should not come in contact with patient, staff or instruments.
3. Close cap and store in a suitable place. Discard container when empty (or sooner, if required by facility internal procedures).

### RECOMMENDED STORAGE CONDITIONS

Store in a clean, dry place at room temperature and away from direct sunlight.



HR Pharmaceuticals, Inc.  
2600 Eastern Blvd., Suite 201  
York, PA 17402 USA  
877-302-1110  
hrpharma.com



Emerge Europe  
Westervoortsedijk 60,  
6827 AT Anhem  
The Netherlands



Imaging Solutions Pty Ltd  
PO Box 3225 Loganholme DC,  
QLD, 4129  
Australia



Nissha Medical Technologies Ltd  
Torbay Business Park, Woodview Rd.  
Paignton, Devon, TQ4 7HP

Nissha Medical Technologies SAS  
23-25 Boulevard de la Paix  
95800 Cergy, France

### SINGLE-USE PACKETS



### MULTI-USE PRODUCTS



Symbols glossary and additional information available at [ecovue.com](http://ecovue.com)

EcoVue is a part of the **HR** family of brands.

HR, EcoVue, FlexPac, and Acclimate are registered trademarks of HR Pharmaceuticals, Inc.

## Sterile Ultrasound Gel

### HV Sterile Ultrasound Gel

### Instructions for Use

#### REF

280	20 g Packet with SafeWrap® Package
280NW	20 g Packet
282	32 g FlexPac® Pouch with SafeWrap® Package
282NW	32 g FlexPac® Pouch
<hr/>	
380	20 g Packet with SafeWrap® Package
380NW	20 g Packet

#### DESCRIPTION

EcoVue® Sterile Ultrasound Gels are sterile, non-irritating, non-sensitizing, non-staining, and water-soluble scanning gels intended for use during sterile ultrasound procedures. The gels are an accessory used to couple sound waves between a patient and medical imaging equipment during ultrasound procedures in a healthcare setting. EcoVue Sterile Ultrasound Gels may be used with all commercially available ultrasound devices.

#### INDICATIONS FOR USE

EcoVue Sterile Ultrasound Gels may be used during ultrasound procedures on all patient populations where sterility is indicated or otherwise desired, or during procedures where cross-contamination is of concern.

#### CONTRAINDICATIONS

None.

#### WARNINGS

- If irritation occurs, discontinue use and consult a physician. NOTICE: Any serious incident that has occurred in relation to this device should be reported to HR Pharmaceuticals and the competent authority in Member State where user is established.
- EcoVue Sterile Ultrasound Gels are intended for single use only. Re-use could result in cross-contamination, risk of infection, or adverse reaction for the patient.
- Caution: Federal law restricts this device to sale by or on the order of a physician or medical practitioner.

#### PRECAUTIONS

- Discard and do not use EcoVue Sterile Ultrasound Gel if package is damaged or opened.
- To reduce the risk of cross-contamination, standard practices for sterile procedures should be used.

#### PRE-USE, HANDLING & INSPECTION

- EcoVue Sterile Ultrasound Gels should be thoroughly inspected prior to use to ensure packaging is not damaged or defective.
- To reduce the risk of cross-contamination, standard practices for sterile procedures should be used.

#### GEL WARMING

EcoVue Sterile Ultrasound Gels, in any packaging format, may be used at room temperature or warmed in an ultrasound gel warmer, such as the Acclimate® Gel Warmer. Refer to the gel warmer manufacturer's operating manual/user manual for warming instructions.

#### STEP-BY-STEP INSTRUCTIONS

1. Remove SafeWrap outer pouch (if applicable).
2. Manually open the packet by tearing along the perforations where indicated.
3. Apply the desired amount of EcoVue directly to the patient or probe as needed.
4. Discard packet after use.

#### RECOMMENDED STORAGE CONDITIONS

Store in a clean, dry place at room temperature and away from direct sunlight.



HR Pharmaceuticals, Inc.  
2600 Eastern Blvd., Suite 201  
York, PA 17402 USA  
877-302-1110  
hrpharma.com



Emerge Europe  
Westervoortsedijk 60,  
6827 AT Anhem  
The Netherlands



Imaging Solutions Pty Ltd  
PO Box 3225 Loganholme DC,  
QLD, 4129  
Australia



Nissha Medical Technologies Ltd  
Torbay Business Park, Woodview Rd.  
Paignton, Devon, TQ4 7HP

Nissha Medical Technologies SAS  
23-25 Boulevard de la Paix  
95800 Cergy, France



Symbols glossary and additional information available at [ecovue.com](http://ecovue.com)

EcoVue is a part of the **HR** family of brands.

HR, EcoVue, FlexPac, SafeWrap, and Acclimate are registered trademarks of HR Pharmaceuticals, Inc.

LBL-0001 Rev. 7 (03/2023)